

**GUTAP**  
**Gutaperka rastvarač**  
**Uputstvo za upotrebu**



Prije upotrebe pažljivo pročitajte upute

**MEDICINSKI UREĐAJ SAMO ZA PROFESIONALNU STOMATOLOŠKU UPOTREBU**

**Polje primjene**

Proizvod za omekšavanje i otapanje gutaperče punjenja. Posebno se preporučuje za:

- Uklanjanje gutaperke tokom re-endo tretmana, u slučaju uklanjanja nategnute gutaperče iz veoma zakrivljenih kanala korena ili veoma dobro kondenzovane i stvrdnute gutaperče i silera.
- punjenje korijenskog kanala metodom jedne tačke ili metodom lateralne kondenzacije.

**Tehnički i medicinski podaci**

Sastav: ulje eukaliptusa 100%

**Pakovanje:**

Boca od jantarnog stakla zatvorena plastičnim čepom sadrži 10 ml supstance. Jedinično pakovanje sa zalijepljenom etiketom upakovano je u kartonsku kutiju zajedno sa uputstvom za upotrebu i kapaljkom.

**Uputstvo za upotrebu**

**Uklanjanje gutaperke iz korijenskih kanala**

- Nanesite 1 - 2 kapi rastvarača u komoru i pomoću endodontske turpije npr. K-fil ISO 25 prodire rastvarač u kanale.
- Nakon otprilike 2 minute uklonite omekšanu gutaperku ručnom endodontskom turpijom, npr. K-File ISO 20 i 15 ili H File tipa.
- U slučaju upotrebe sljedeće doze proizvoda, implementirajte je u djelomično očišćeni korijenski kanal. Irigirajte korijenske kanale naizmjenično sa 2 3% natrijum hipohlorita u količini od 2 ml, koristeći ejektor za pljuvačku.
- Nemojte koristiti rastvarač kada endodontska turpija dosegne 2/3 dužine korijenskog kanala.
- Uklonite materijal za punjenje i irigirajte korijenski kanal dok tekućina za pranje ne postane kristalno bistra.

**Punjenje korijenskog kanala metodom jedne tačke ili metodom lateralne kondenzacije**

- Nakon kompletne preparacije korijenskog kanala odaberite i isprobajte master point, koji bi trebao biti iste veličine kao MAF (Master Apical File).
- Gutaperča vrh treba da bude iste dužine ili 1 mm kraći.
- Umočite vrh gutaperče u rastvarač.
- Ubacite pripremljenu gutaperku vrh u korijenski kanal kako bi se malo zaklinjao i odupirao.
- Uklonite gutaperku šiljak sa sealera kanala pomoću ručne endodontske turpije ili mašinskih sistema ili podmažite gutaperku tačku sa sealerom i još jednom unesite kanal.
- Gutaperka vrh treba da se zaglavi u apikalni foramen.
- Na kraju implementirajte materijal koristeći metodu bočne kondenzacije.

**Upozorenja, mjere opreza i kontraindikacije**

- ☞ Nemojte koristiti proizvod u slučaju preosjetljivosti na ulje eukaliptusa.
- ☞ Nemojte koristiti rastvarač izvan vrha korena.
- ☞ Ne dozvolite da se rastvarač infiltrira izvan vrha korena.
- ☞ Preporučljivo je koristiti koferdam tokom nanošenja proizvoda. Koristite zaštitne mere (naočare, rukavice). Izbegavati kontakt sa očima, mekim tkivima usne duplje i kožom.



Gutanje i difuzija kroz zračne prolaze može uzrokovati smrt.

**AKO SE PROGUTA:** Odmah pozvati **CENTAR ZA TROVANJE** ili ljekara. NE izazivati povraćanje.

**AKO DOSPE NA KOŽU** (ili kosu): Odmah skinuti/skinuti svu kontaminiranu odjeću. Isperite kožu vodom/tuširajte se.

**Lako zapaljiva tečnost i para.**

**Toksično za vodeni svijet s dugotrajnim efektima.**

Odložiti sadržaj/kontejner u skladu sa zahtjevima korištenja.

**Mesto i uslovi skladištenja**

Držite posudu dobro zatvorenu. **Čuvati van domašaja djece.** Štiti od vlage.

*Store in temperature below 25°C.*

*Keep away from sunlight.*

*The lot number is given on the outer and primary package.*



*The expiry date is given on the outer and primary package. Do not use after expiry date. After opening the expiry date does not change provided that the container is tightly closed after each use.*



Producent / Manufacturer:

CHEMA-ELEKTROMET

Spółdzielnia Pracy ul. Przemysłowa 9

35-105 Rzeszów, Polska [www.chema.rzeszow.pl](http://www.chema.rzeszow.pl)

[chema@chema.rzeszow.pl](mailto:chema@chema.rzeszow.pl)

Verified : 2021-05

